Jay Foam Cushion Products

User Instruction Manual & Warranty

Jay Foam Kussens

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Jay Skumdynor Produkter

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Jay Skum Pude Produkter

Brugsanvisning Manuel og Garanti

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ENGLISH**DUTCH****SWEDISH****FINNISH****DANISH**

- Adjust lumbar support height so that maximum thickness of lumbar support is at your waist or belt line.
- Attach the lumbar support by looping one Velcro® strap attachment over the back of the chair, under the push handle and back around the front of the chair. The hook Velcro on the back of the strap should secure to the loop Velcro on the front of the strap.
- Repeat the attachment process on the opposite side, making sure that the lumbar support is centered on the wheelchair.
- If additional support is required, a 1/2" (1cm) thick foam insert can be placed inside the cover behind the lumbar support. This acts as a shim to increase the thickness of the lumbar support.

Note: Junior size lumbar supports attach with an elastic strap and buckle instead of Velcro® straps.

Hint: Some people find that varying the height of the lumbar support during the course of the day helps relieve sitting fatigue. They also find that placing the cushion about 1" (2.5 cm) further forward in the wheelchair makes it more comfortable to use with the lumbar support. If your cushion did not include a lumbar support but you would like its benefits, contact an authorized supplier for a free demonstration.

Easy maintenance

Monthly cleaning and regular maintenance may help extend the life of your cushion. During cleaning, component inspection is recommended. Check the cover for tears and excessive wear.

To clean cover

Remove the cover from the base. Turn the cover inside out and machine wash in warm water and drip or tumble dry on low heat. No ironing required.

Note: Do not dry clean covers or use industrial washers and dryers to clean cover. Do not steam autoclave. Do not bleach.

After washing, fit cover back over foam base. Ensure the back of the cover matches up to the back of the foam base.

Note: If utilized in an institutional setting, write the resident's name with a permanent marker on the content label of the covers. This will help to ensure the return of the covers after washing.

Use only Jay-designed covers in conjunction with Jay foam cushions.

To clean foam base

Remove the cover from the foam base. Wipe with warm water and soap. Wipe off with a clean cloth. Avoid complete submersion of the foam base in water.

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ENGLISH**DUTCH****SWEDISH****FINNISH****DANISH****LIMITED WARRANTY**

Sunrise Medical guarantees its products to be free from defects in materials and workmanship for a specific period of time from date of purchase. Determine which cushion was purchased and use the following information to determine which warranty applies.

- The Jay lumbar support is guaranteed for one year from date of purchase.
- The Jay Basic Cushion is guaranteed for one year from date of purchase.
- The Jay Triad Cushion is guaranteed for two years from date of purchase.
- The Jay Combi Cushion and Jay Soft Combi Cushions are guaranteed for five years from date of purchase. Sunrise Medical will, at its option, repair or replace your cushion without charge. The warranty does not apply to, punctures, tears, or burns, nor to the cushion's cover. Claims and repairs should be processed through the nearest authorized equipment supplier. Except for express warranties made herein, all other warranties including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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**ENGLISH**

Sunrise Medical
Home Healthcare Group
Mobility Products Division
7477 East Dry Creek Parkway
Longmont, Colorado 80503
USA
(303) 218-4500

DUTCH

Sunrise Medical BV
Importeur
Pascalbaan 3
3439 MP Nieuwegein
tel: 030-6082100

SWEDISH

Sunrise Medical AB
Britta Sahlgrens gata 8A
SE-421 31 VÄSTRA FRÖLUNDA

FINNISH

Merivaara Instrumentarium
Puusteinintie 2
Lahti FINLAND
SF 15150

DANISH

G2 Wheelchair and Gear
Jegstrupvej 56
DK 8361 Hasselager DENMARK
DK 8361

This product is manufactured to comply with the 'Medical Device Directive' 93/42/EEC.