


# THE EQUAGEL™ PROTECTOR CUSHION

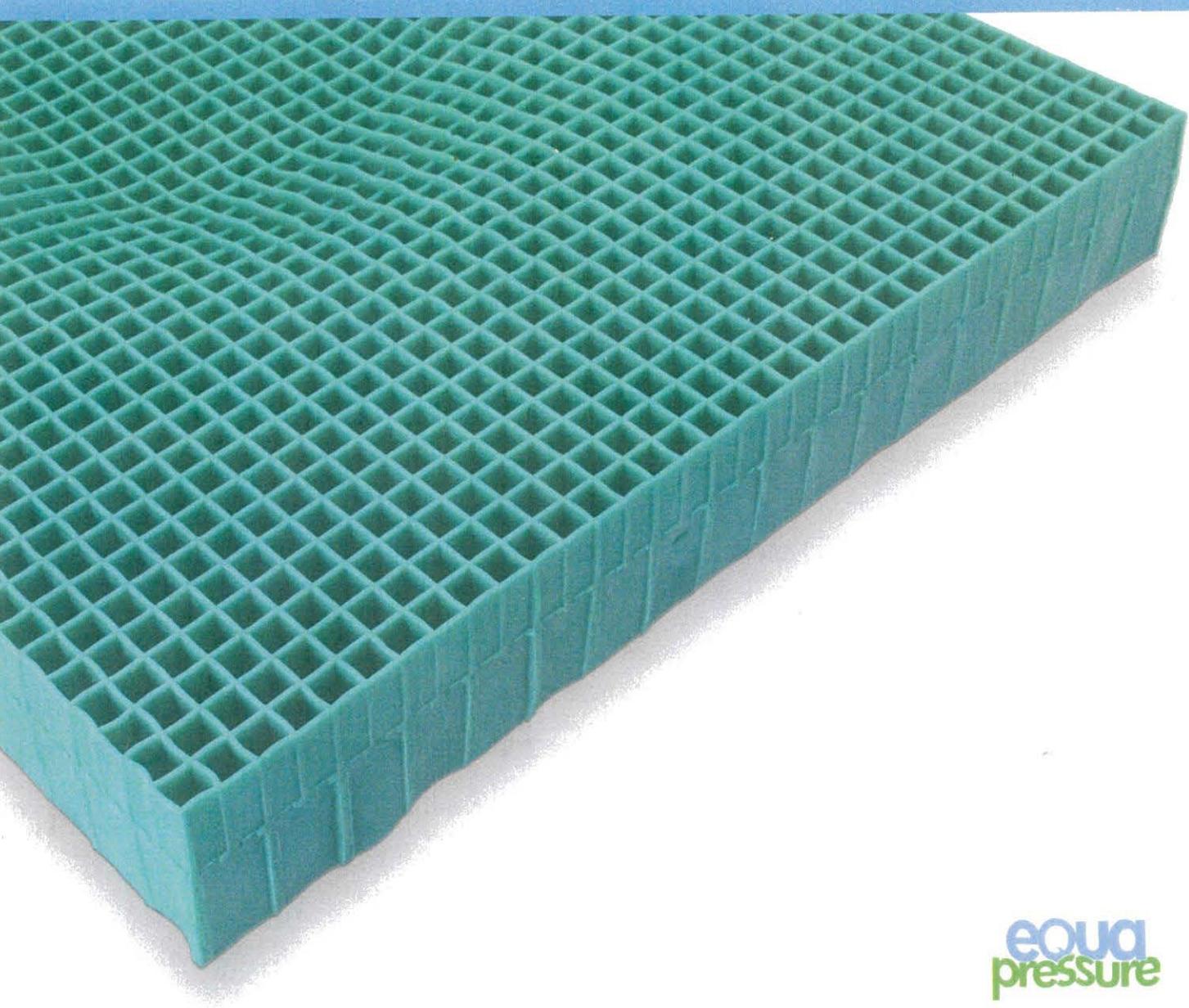

- |  |  |  |  |
|--|--|--|--|
| <p><b>X 16"W x 16" D x 2.5" H</b><br/>40.5 cm x 40.5 cm x 6.5 cm</p> | <p><b>□ 18"W x 16" D x 2.5" H</b><br/>45.5 cm x 40.5 cm x 6.5 cm</p> | <p><b>□ 20"W x 16" D x 2.5" H</b><br/>51.0 cm x 40.5 cm x 6.5 cm</p> | <p><b>□ 22"W x 16" D x 2.5" H</b><br/>56.0 cm x 40.5 cm x 6.5 cm</p> |
| <p><b>□ 16"W x 18" D x 2.5" H</b><br/>40.5 cm x 45.5 cm x 6.5 cm</p> | <p><b>□ 18"W x 18" D x 2.5" H</b><br/>45.5 cm x 45.5 cm x 6.5 cm</p> | <p><b>□ 20"W x 18" D x 2.5" H</b><br/>51.0 cm x 45.5 cm x 6.5 cm</p> | <p><b>□ 22"W x 18" D x 2.5" H</b><br/>56.0 cm x 45.5 cm x 6.5 cm</p> |

40-81-68277



## WHAT IS EQUAGEL™?

Arranged in a dual-layer grid pattern, EquaGel™ is engineered to provide both optimal pressure relief and support through column-buckling technology. Add to that the outstanding tensile and compression strength, unparalleled durability, and temperature-neutral properties, EquaGel™ cushions are the go-to choice for long-term sitting care.

## HOW DOES IT WORK?


The science of EquaGel™ is based on its column-buckling properties. Made with a dry polymer gel, each grid wall has a low weight threshold — when exceeded, the wall instantly buckles, passing the weight to surrounding walls for support. This weight distribution continues into each cell until the pressure is equalized.

High-pressure areas, such as the ischial tuberosities and coccyx/sacrum, activate the column-buckling pressure relief, providing an almost zero-gravity feeling. The low-pressure thighs and buttocks are supported as the weight is evenly distributed, which aids the overall pressure relief.

## WHY IS IT DUAL-LAYERED?

The two-stage buckling design allows the user to experience the most pressure relief and support possible. The soft micro-grid top layer is extra sensitive to pressure, allowing maximum alleviation. The wider bottom grid features thicker walls that hold the user up in good posture, and prevents bottoming out.

Select EquaGel™ cushions also feature a pelvic contour that mimics a natural sitting shape, promoting even deeper pressure relief.

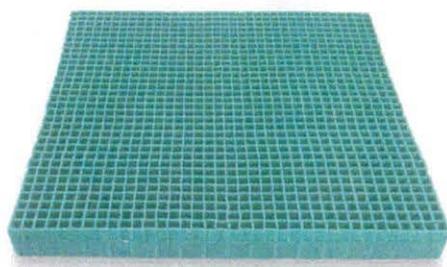

## THE EQUAGEL™ STRAIGHT COMFORT CUSHION BASIC PRESSURE THERAPY

**Material:** 100% Dual-layer EquaGel™

**Thickness:** 1.75" (4.5 cm)


**Top Layer:** Thin-wall gel with a micro-grid for a comfort layer.

**Bottom layer:** Thick-wall gel with a wide grid for deep posture support.

**Cover:** Fluid-resistant, vapor-permeable, and flame-resistant.  
46% Polyester, 36% PVC, 18% Polyurethane Fiber

## AVAILABLE SIZES

16" x 16"	18" x 16"	20" x 16"	22" x 16"
16" x 18"	18" x 18"	20" x 18"	22" x 18"
16" x 20"	18" x 20"	20" x 20"	22" x 20"

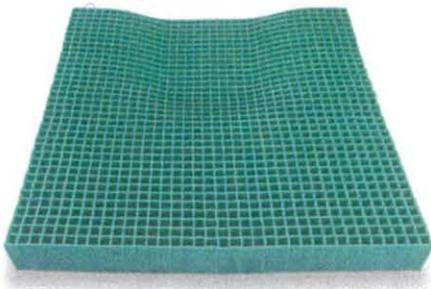

## THE EQUAGEL™ GENERAL CUSHION

### MODERATE PRESSURE THERAPY

**Material:** 100% Dual-layer EquaGel™

**Thickness:** 2" (5.1 cm)

**Top Layer:** Thin-wall comfort gel with a micro-grid for a comfort layer.

**Bottom layer:** Thick-wall support gel with a wide grid for deep posture support.

**Contoured Pelvic Area:** Designed to more evenly distribute the user's weight than flat cushions, while retaining 1.25" of buckling gel under the pressure points.

**Cover:** Fluid-resistant, vapor-permeable, and flame-resistant.  
46% Polyester, 36% PVC, 18% Polyurethane Fiber

### AVAILABLE SIZES

16" x 16"	18" x 16"	20" x 16"	22" x 16"
16" x 18"	18" x 18"	20" x 18"	22" x 18"
16" x 20"	18" x 20"	20" x 20"	22" x 20"



## THE EQUAGEL™ PROTECTOR CUSHION

### ADVANCED PRESSURE THERAPY

**Material:** 100% Dual-layer EquaGel™

**Thickness:** 2.5" (6.4 cm)

**Top Layer:** Thin-wall comfort gel with a micro-grid for a comfort layer.

**Bottom layer:** Thick-wall support gel with a wide grid for deep posture support.

**Contoured Pelvic Area:** Designed to more evenly distribute the user's weight than flat cushions, while retaining 1.75" of buckling gel under the pressure points (e.g. ischial tuberosities and coccyx/sacrum).

**Cover:** Fluid-resistant, vapor-permeable, and flame-resistant.  
46% Polyester, 36% PVC, 18% Polyurethane Fiber

### AVAILABLE SIZES

16" x 16"	18" x 16"	20" x 16"	22" x 16"
16" x 18"	18" x 18"	20" x 18"	22" x 18"



## THE EQUAGEL™ ADJUSTABLE PROTECTOR CUSHION

### CUSTOMIZABLE ADVANCED PRESSURE THERAPY

**Material:** 100% Dual-layer EquaGel™

**Thickness:** 2.5" (6.4 cm)

**Top Layer:** Thin-wall comfort gel with a micro-grid for a comfort layer.

**Bottom layer:** Thick-wall support gel with a wide grid for deep posture support.

**Contoured Pelvic Area:** Designed to more evenly distribute the user's weight than flat cushions, while retaining 1.75" of buckling gel under the pressure points (e.g. ischial tuberosities and coccyx/sacrum).

**Adjustable Inserts:** Included are two gel inserts (.25" and .5" thick) that can increase the amount of therapeutic gel under the pelvis from 1.75" to:

2" (use .25" insert)

2¼" (use .5" insert)

2½" (use .25" + .5" inserts)

To adjust the cushion, slip the gel insert(s) into the fabric pocket inside the cover, beneath the gel cushion. Place the insert(s) with the rounded edges going into the fabric sleeve first, grid side up.

**Cover:** Fluid-resistant, vapor-permeable, and flame-resistant.  
46% Polyester, 36% PVC, 18% Polyurethane Fiber

### AVAILABLE SIZES

16" x 16"

18" x 16"

20" x 16"

22" x 16"

16" x 18"

18" x 18"

20" x 18"

22" x 18"

## CARE INSTRUCTIONS

**Cover:** Machine wash cold on a gentle cycle and tumble dry with NO HEAT.

\*Washing or drying with heat may crack the non-slip fabric and void the cover's warranty.

**Gel Cushion:** Handwash with warm water and gentle detergent (e.g. dish soap), then rinse with warm water only. Lay flat to air dry.

**Heat Warning:** If your cushion gets above 105° F, do not sit on or fold it, as it could warp the EquaGel™. Simply let it lie flat to cool before use. If you leave your cushion in the car on a hot day, keep it cooler by placing it under the seat or putting a towel over it.

EquaPressure and EquaGel are trademarks of EdiZONE, LLC of Alpine, Utah, USA. EquaPressure, LLC's EquaGel™ cushions are Protected by U.S. Patents 5,749,111, 6,026,527, 6,413,458, 7,060,213, 7,964,664, and, only if the screed molding process is used 7,666,341.

### EQUAPRESSURE CUSHIONS ARE WARRANTIED AGAINST DEFECTS FOR 18 MONTHS.

While EquaGel cushions are intended to aid the treatment of pressure sores, only a medical professional can determine your treatment needs and the effectiveness of the cushion.